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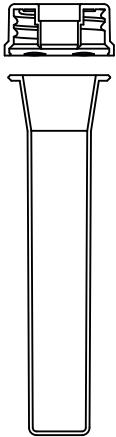
# Holed screw cap and protective sheath

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**Material:** PE

For bottle threads PP25, PP28 and PP31, compatible for all pipette sizes.

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## Handling procedure for dosing pipette with holed screw cap and protective sheath

1. Open the bottle and remove the sealing cap.
2. Using thumb pressure, detach the protective sheath from screw cap (Fig.1) and then pull out the pipette with screw cap (Fig.2).
3. Screw the holed screw cap with mounted pipette onto the bottle. The pipette is now ready for use and remains screwed on the bottle.
4. At the start of the measuring procedure the transparent dosing body as well as the white plunger must be in the bottommost position. To measure the dosing quantity use one hand to hold the bottom transparent dosing body and the other hand to pull on the top white plunger until you can read the desired quantity in ml or the body weight in kg (Fig.3).
5. Pull the pipette by the dosing body (bottom transparent ring) out of the bottle (Fig.4).
6. Put the quantity removed by the dosing pipette onto a spoon by pressing down on the plunger (white ring) or empty into a glass with some liquid and give it to the patient. The drug can also be given directly from the pipette into the patient's mouth.
7. Push the empty dosing pipette into the holed screw cap of the bottle until it is used next time.