Adapter für Dosierpipetten

Adaptor for dosing pipettes

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Material: PE

For bottleneck PP 28, available for glass-, PE- and PET bottles. Suitable for all elm-plastic pipettes with flat tip and with elm-tip.

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Using the adaptor can be of advantage in the following ways:

By using the adaptor a length-independency between bottle and pipette is achieved. It gives an opportunity to use a relatively small pipette in combination with a larger bottle. The adaptor can be mounted in the bottleneck either during the filling process or by the patient. Furthermore a contamination of the bottle content is avoided by using the adaptor as there is no contact between pipette surface and the drug. Due to the special design of the adaptor a complete emptying of the bottle is possible.

Handling procedure:

The pipette is put in the adaptor in the upright position and then turned upside down together with the bottle (Fig. 1). In this position the medicine is dosed. Afterwards the bottle is turned in the upright position again and the dosed pipette can be removed (Fig. 2). The bottle is sealed with the inserted adaptor by using a common closure.

Abb 1
Fig. 1

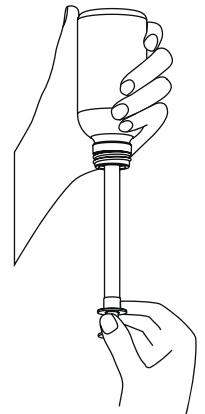
Abb. 2
Fig. 2